



Setting Up & Using Predictable (5.0.7)

Learn more about **Predictable** products
<https://www.therapy-box.co.uk/predictable>

LEARNING OBJECTIVES

By the end of the training, you will be able to:

- Identify individuals who would benefit from this app
- Set-up and personalize vocabulary
- Maximize use of app for phone communication

PREDICTABLE IN ACTION

CONSIDERATIONS FOR SELECTION


- Has developed or retained effective literacy skills
- Comfortable with text-based communication
- May need alternative or customized keyboard
- Uses phrases to communicate

SETTING UP A USER




1. Tap **User** or **Register Now**.
2. Type Name, email and password confirmation, then tap **Sign Up**.
3. Access the email from Therapy Box, select **Activate**.
4. **Sign in** Predictable.
5. Use the **Setup Wizard**.
6. Tap **Go to Predictable**.

NAVIGATION

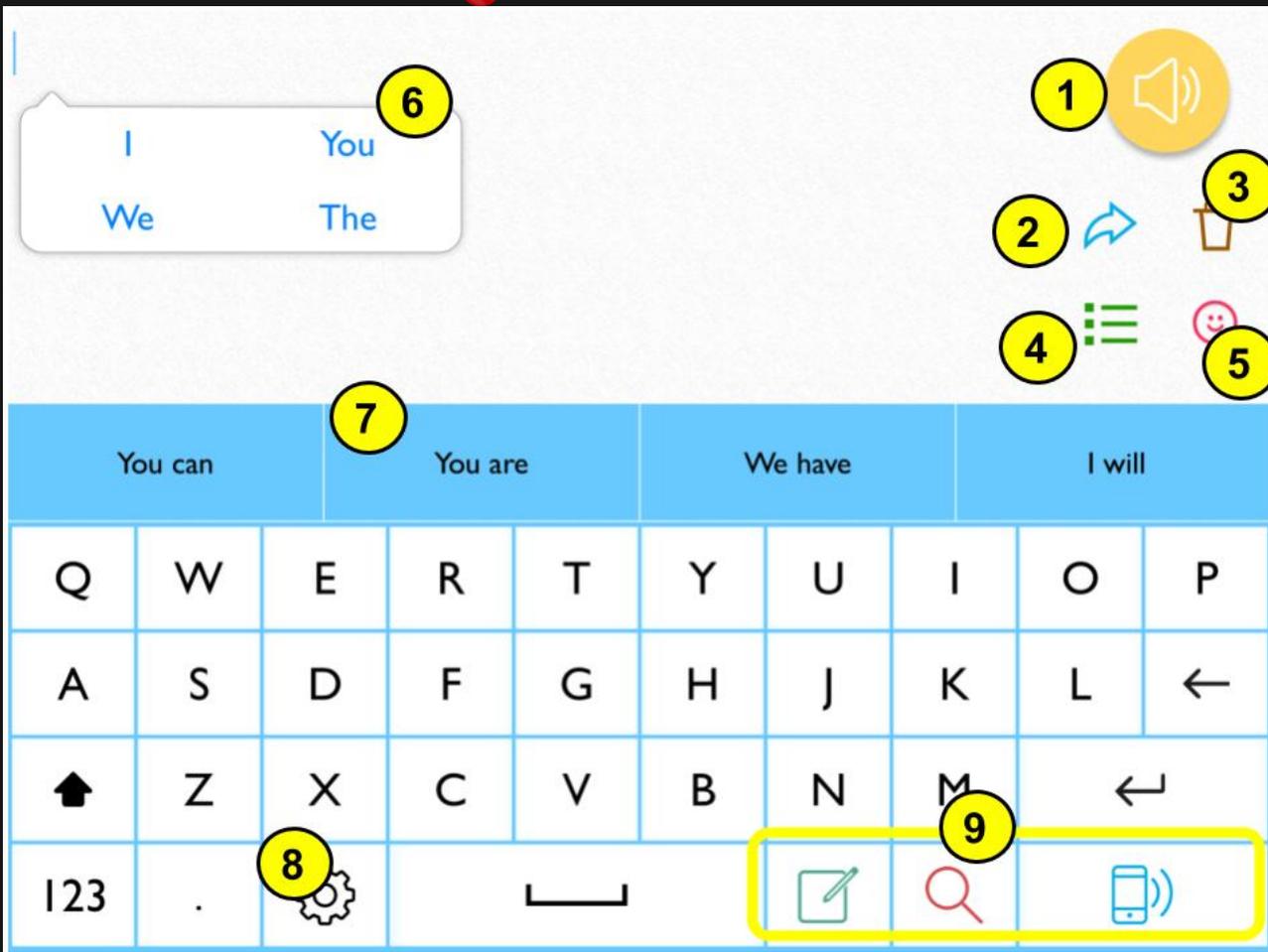


1. Speak button
2. Use Key (quick key)
3. Delete (quick key)
4. Categories (quick key)
5. Emotes (quick key)
6. Bubble prediction
7. Strip prediction
8. Settings key
9. Keyboard Feature Keys

SOUND SETTINGS: SELECTING A VOICE

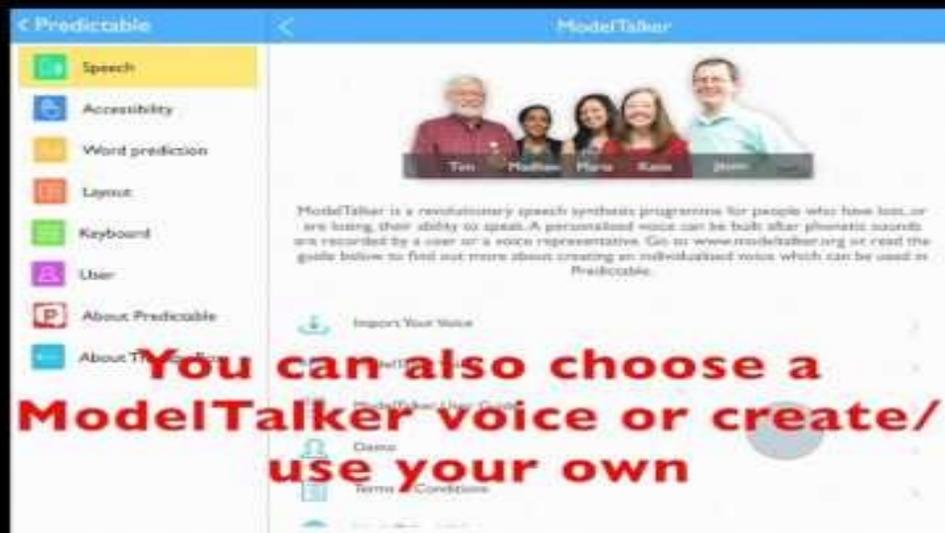


1. Tap the **gear** icon to open the settings.
2. Tap **Speech**.
3. Use an iOS voice from **iOS**, or an enhanced free voice from Predictable's voices **Nuance**.
4. Tap **Predictable**.

ADDING AN iOS VOICE

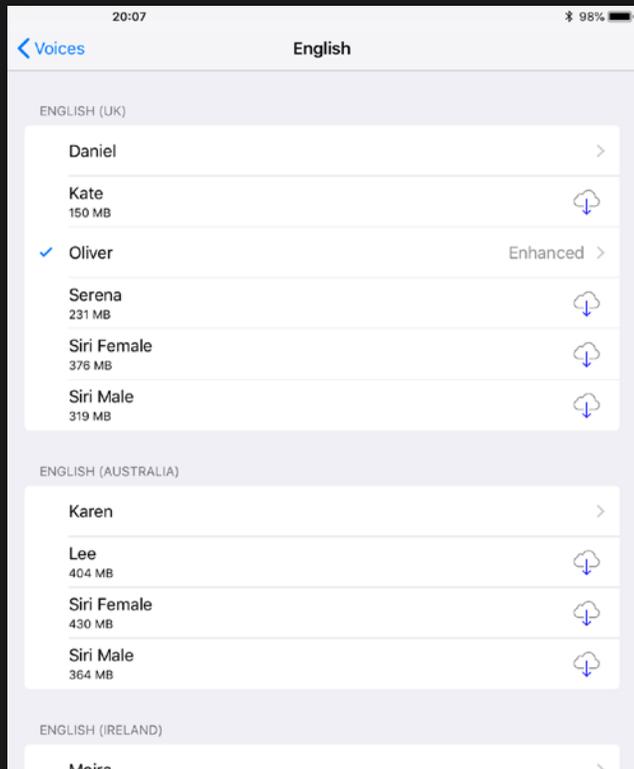


1. Within iPad Settings App, Tap **Accessibility**.
2. Tap **Spoken Content**.
3. Tap **Voices**.
4. Select desired language.
5. Tap the **cloud** icon to download and preview the voice.
6. Tap the iPad's **Home button**.

MODIFYING VOICE QUALITIES

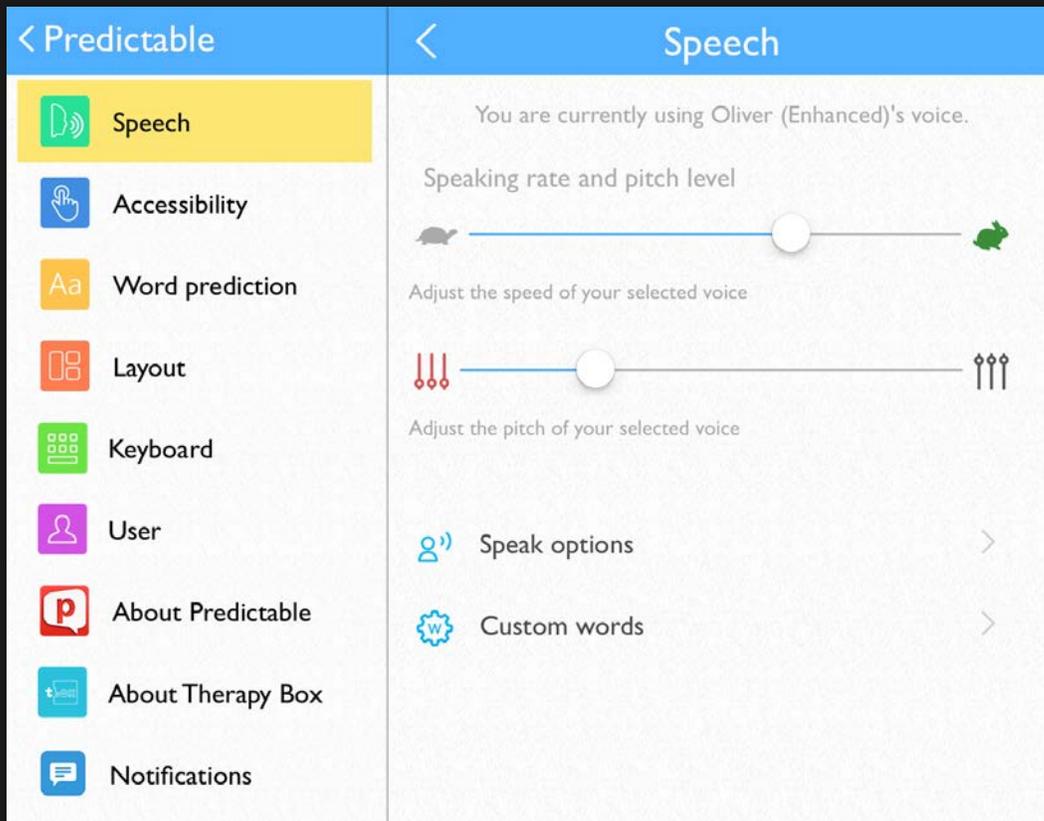


1. Tap the **gear** icon.
2. Tap **Speech**
3. Tap the **gear** icon with arrows.
4. Use **sliders** to change the **speed** and **pitch** the voice speaks.
5. Tap **Speak Options** to personalize when Predictable will speak.
6. Tap **Custom Words** to personalize pronunciation of word.
7. Tap **Predictable**.

SOUND SETTINGS: TURNING BUTTON CLICKS ON AND OFF

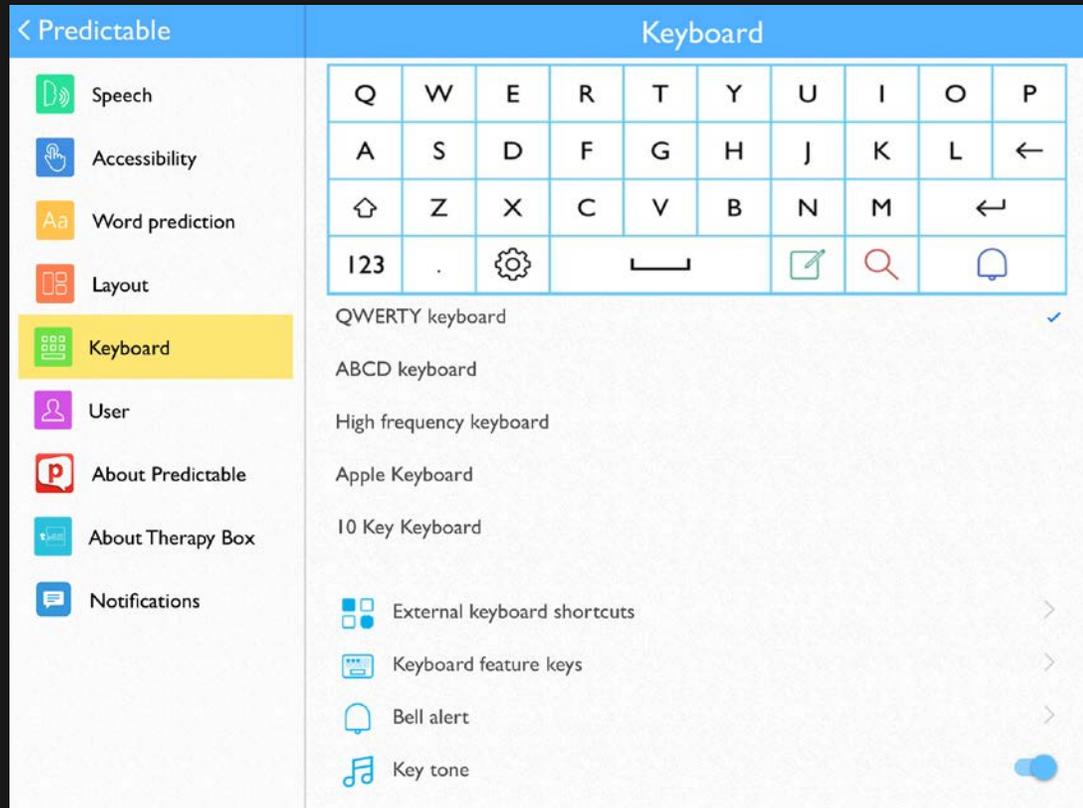


1. Tap the **gear** icon to open the settings.
2. Tap **Keyboard**.
3. Toggle **Key Tone** to blue.
4. When toggled to grey no **Key Tone** will sound when buttons are selected.

RATE ENHANCEMENT: QUICK KEYS

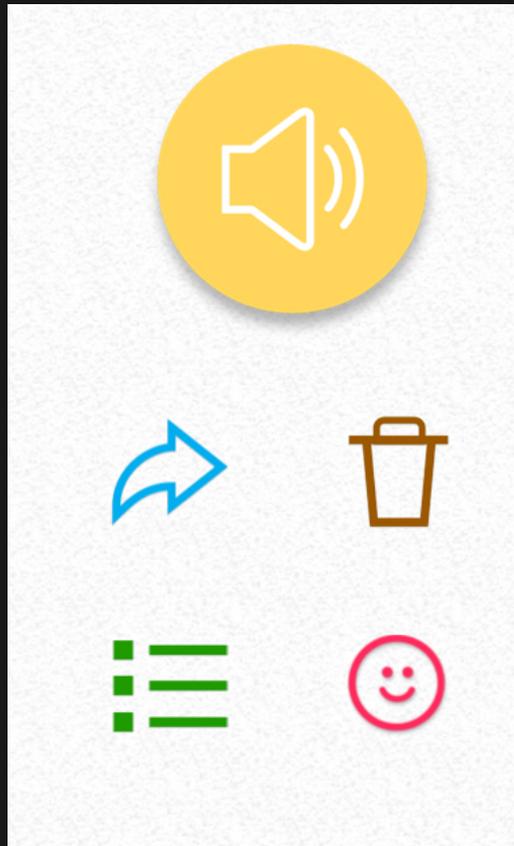


- Use
- Delete
- Categories
- Emotes

RATE ENHANCEMENT: WORD PREDICTION

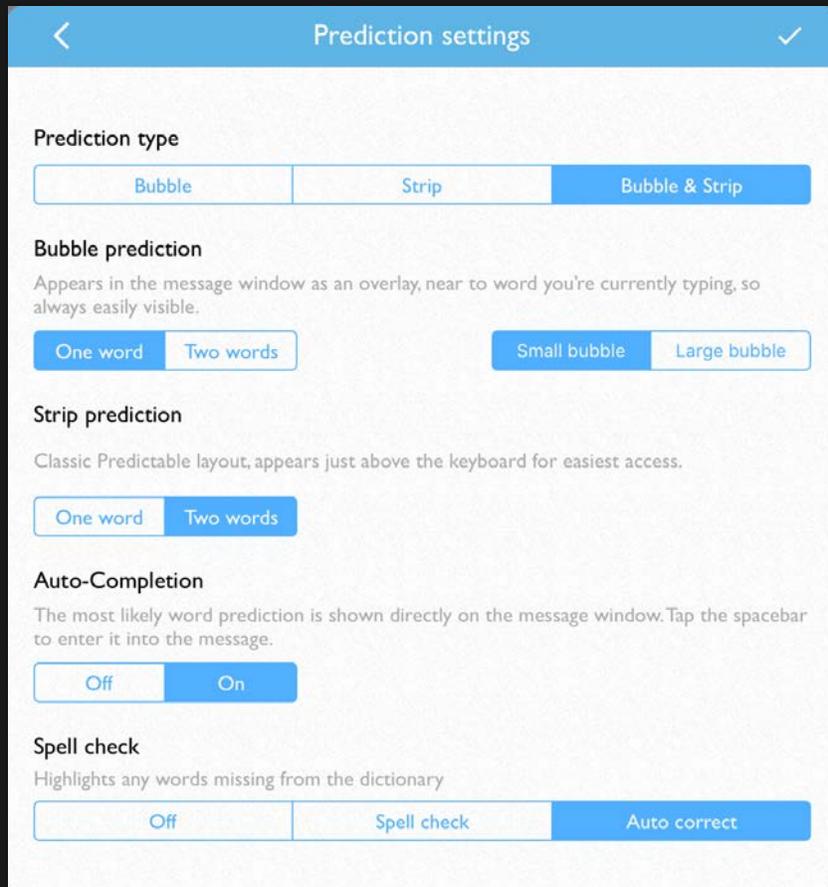


1. Tap the **gear** icon.
2. Tap **Word Prediction**.
3. Tap **Prediction Type**.
4. Select **Bubble** or **Sentence** prediction, or set to **Both**.
5. Tap **Predictable**.

RATE ENHANCEMENT: DYSLEXIA SUPPORT

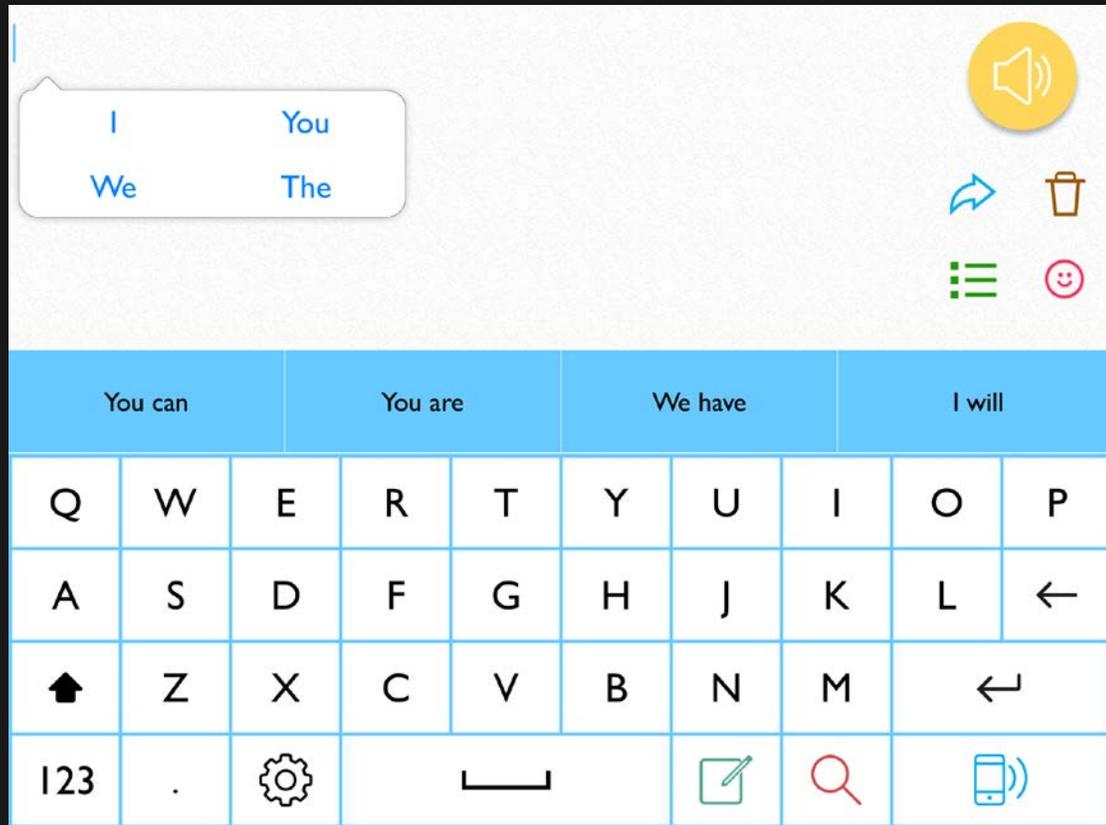


1. Tap the **gear** icon.
2. Tap **Word Prediction**.
3. Tap **Prediction Type**.
4. Toggle **Auto-Completion** to on.
5. Change **Spell check** options to **highlight** misspelled words or **to Auto-correct**.
6. Tap **Predictable**.

RATE ENHANCEMENT: FEATURE KEYS

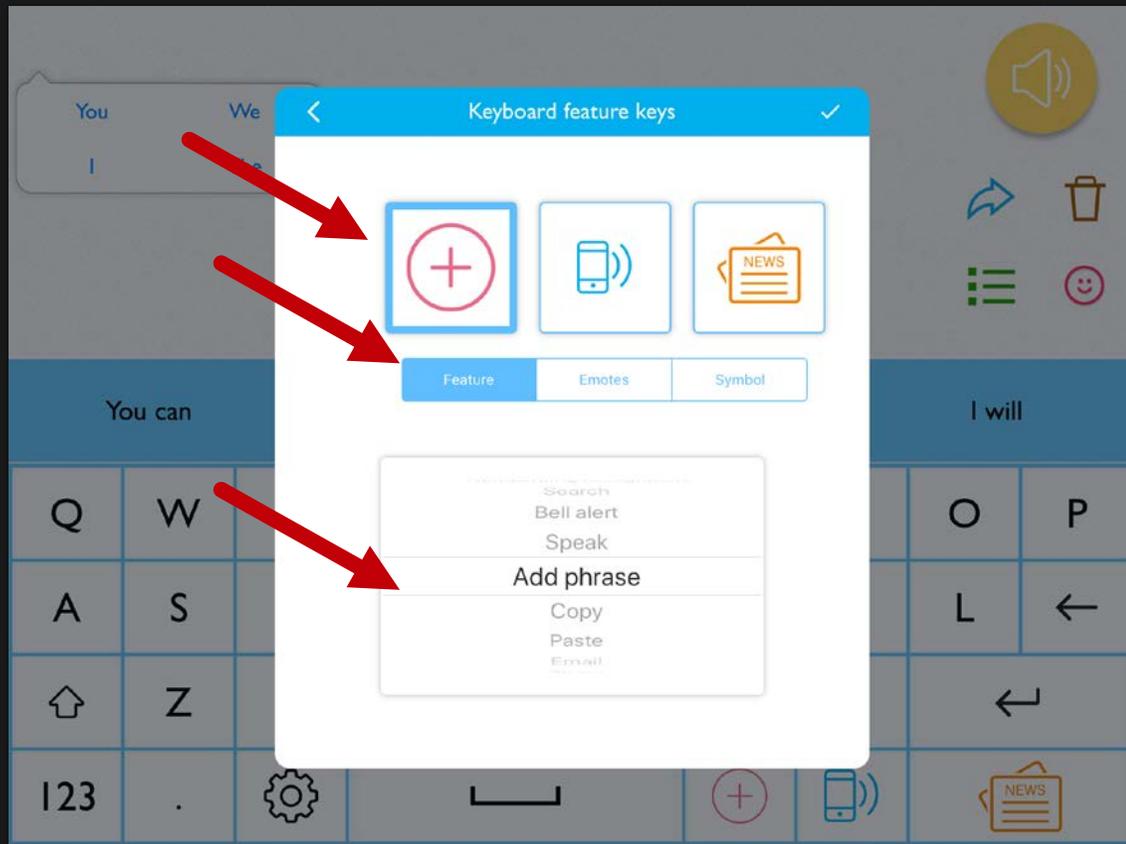


1. Tap the **gear** icon.
2. Tap **Keyboard**.
3. Tap **Keyboard Feature Keys**.
4. Tap on a **Keyboard feature image** icons, then select a menu: **Feature**, **Emotes**, or **Symbol**.
5. Scroll and select the **feature**.
6. Tap **Predictable**.

RATE ENHANCEMENT: HANDWRITING RECOGNITION

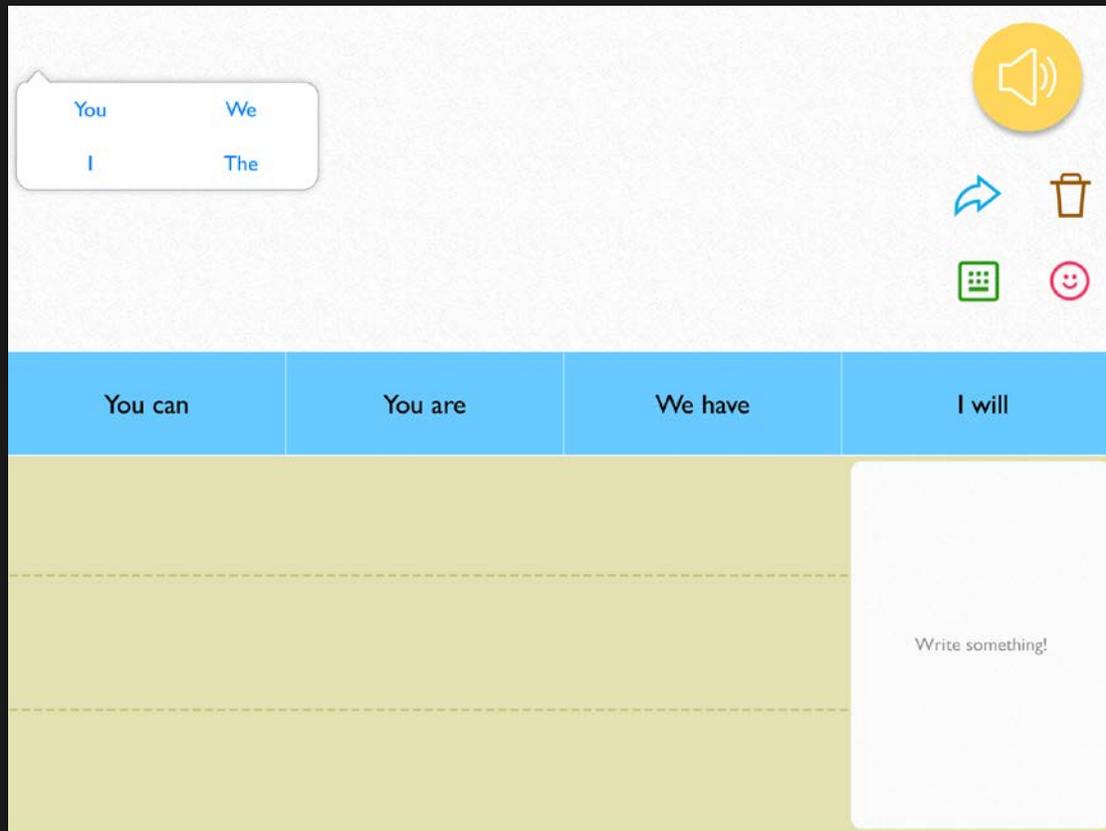


1. Tap the green **Pen and Paper** icon.
2. **Handwrite** letter/s.
3. Use the writing pad prediction or tap on the brown **Pencil Eraser** icon to rewrite letter/s.
4. Tap the **Keyboard** icon in the message window.

SETTING UP THE KEYBOARD

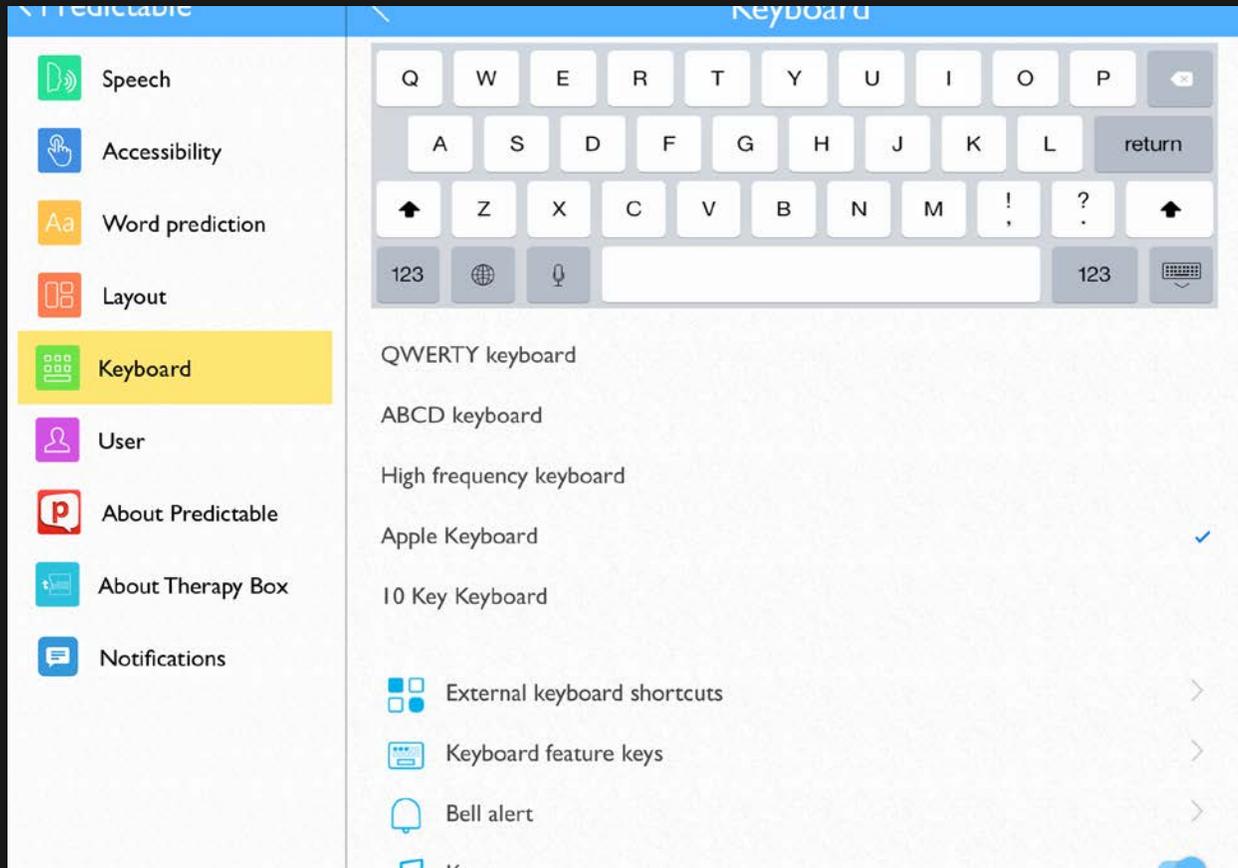

1. Tap the **gear** icon.
2. Tap **Keyboard**.
3. Select a keyboard.
4. Tap **Predictable**.

USING AND ADDING EMOTES

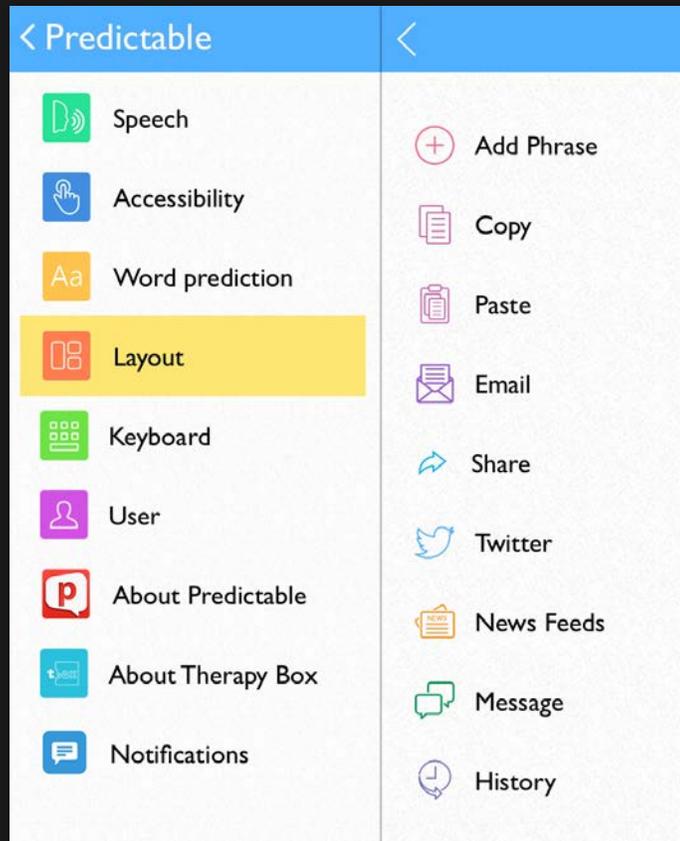

1. Tap the **gear** icon to open the settings
2. Tap **Layout**.
3. Tap **Manage Emotes**.
4. Tap **Create new emote**, then type an emote **name**.
5. Select **Record Now** or **Choose from Audio Library**.
6. Select an **image** for the emote.
7. Tap on an **Emote** and select up to nine.
8. Tap **Predictable**.

ACCESSIBILITY: FLOOR HOLD




1. Tap the **gear** icon to open the settings.
2. Tap **Accessibility**.
3. Tap **Floor hold**.
4. Select the floor hold **option**.
5. Use the **slider** to change how frequent the Floor hold will speak.
6. Tap **Add new floor hold phrase** to add a new phrase.
7. Tap **Predictable**.

CUSTOMIZING: CHANGING THE APPEARANCE


1. Tap the **gear** icon to open the settings.
2. Tap **Layout**.
3. Tap **Appearance**.
4. Personalize settings.
5. Tap **Predictable**.

EDIT AN EXISTING CATEGORY: NAME

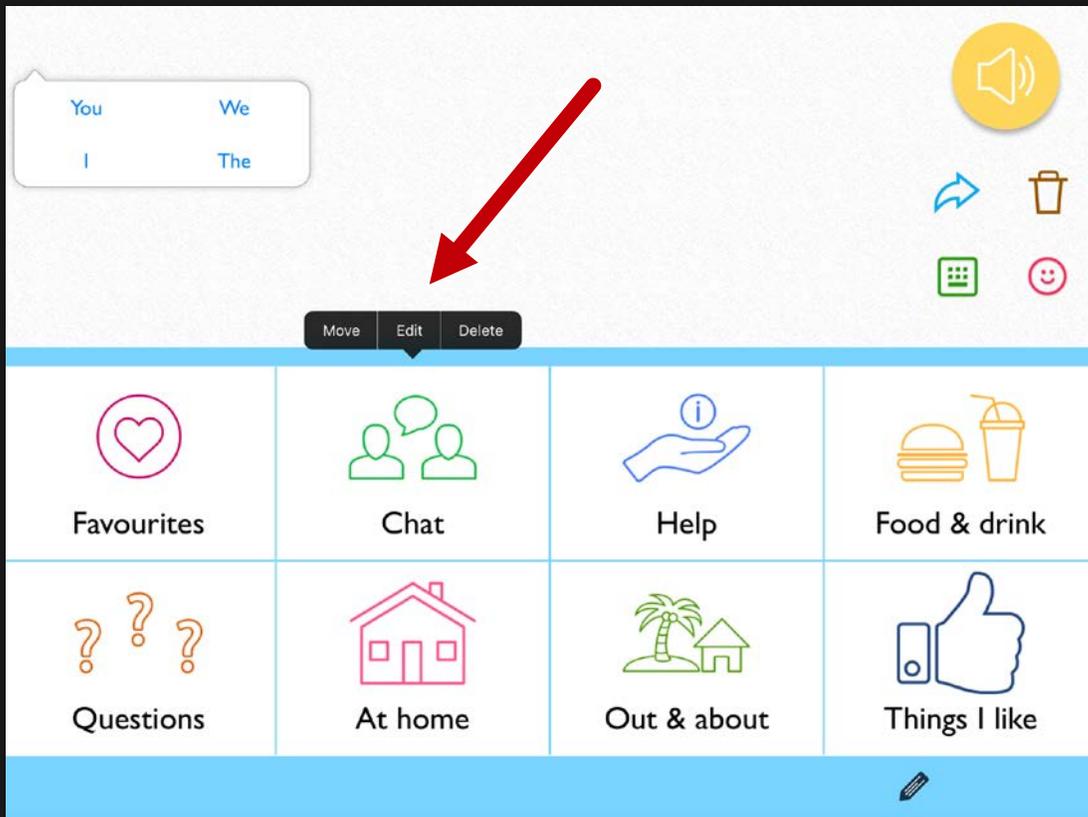


1. Tap the **Phrases Quick Key** icon to open the categories section.
2. Tap the **Pencil** icon to allow editing.
3. Tap on the **Category** to edit.
4. Three options will appear: **Move**, **Edit**, and **Delete**.
5. Tap **Edit**, then customize.
6. Tap **Predictable**, tap **Save**.

EDITING AN EXISTING PHRASE: LABEL AND MESSAGE

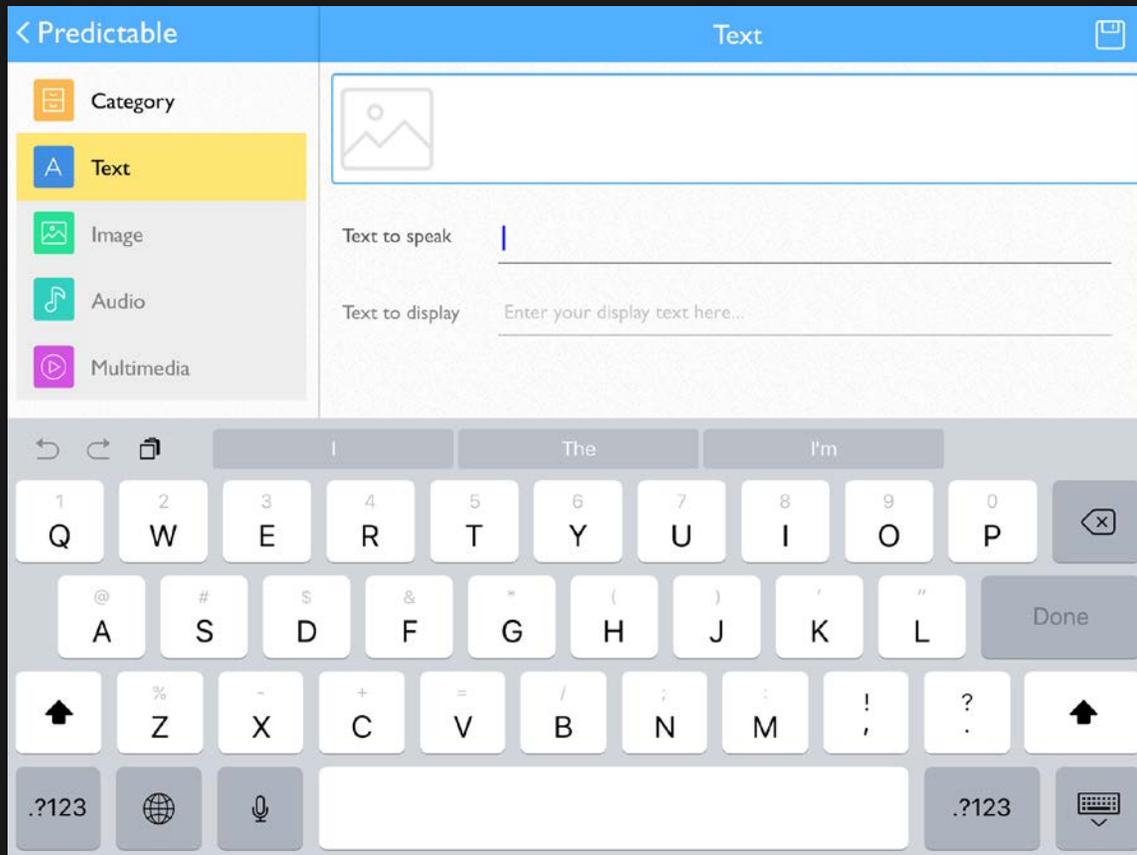

1. Tap the **Phrases Quick Key** icon to open the categories section.
2. Tap on the **Category** to edit.
3. Tap the **Pencil** icon, then tap a **phrase**.
4. Three options will appear: **Move**, **Edit**, and **Delete**.
5. Tap **Edit**, then customize.
6. Tap **Predictable**, tap **Save**.

EDITING A PHRASE OR CATEGORY IMAGE

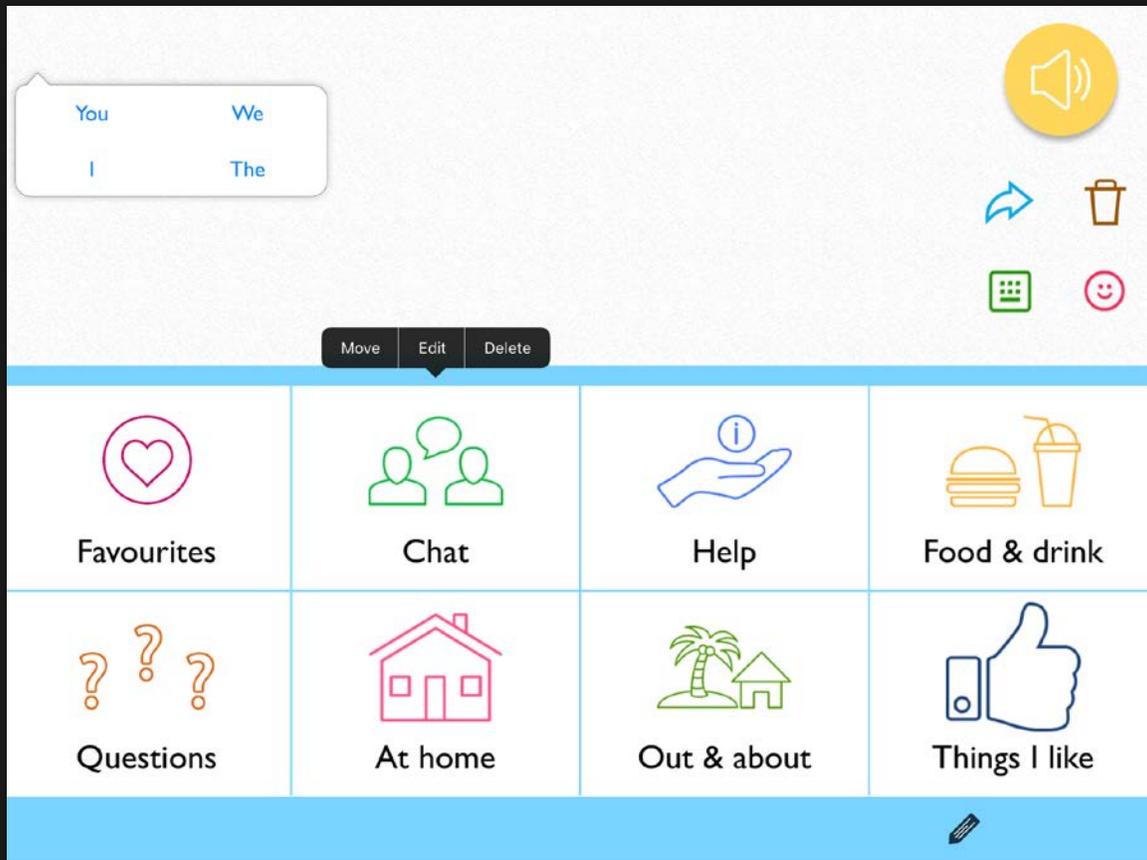

1. Choose an image from the camera roll, take a live picture, or add a picture from Predictable's image library.
2. Tap the **disc** icon to save.
3. Select **Predictable** in the upper left hand corner to exit edit mode.
4. Tap the **Keyboard** icon to return to the keyboard.

COMMUNICATION OVER ANY SPEAKERPHONE



IN-PERSON VS. DISTANCE COMMUNICATION



YOU CAN HEAR ME & SEE ME




- Signs
- Facial expressions
- Gestures
- Vocalizations
- Context
- Communication device

YOU CAN ONLY HEAR ME




- Vocalizations
- Context
- Communication device

PURPOSE OF A PHONE SCRIPT

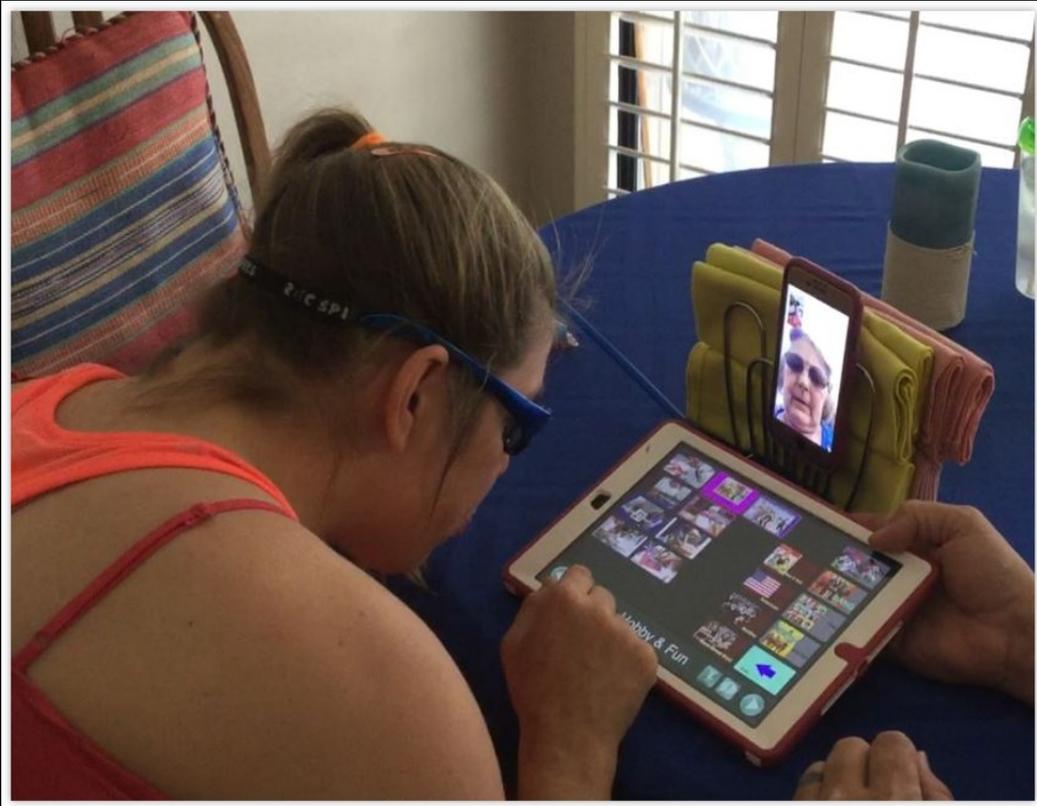


- Speed up communication
- Get your main point across accurately
- Control the conversation
- Repair breakdowns

CONTENTS OF A PHONE SCRIPT

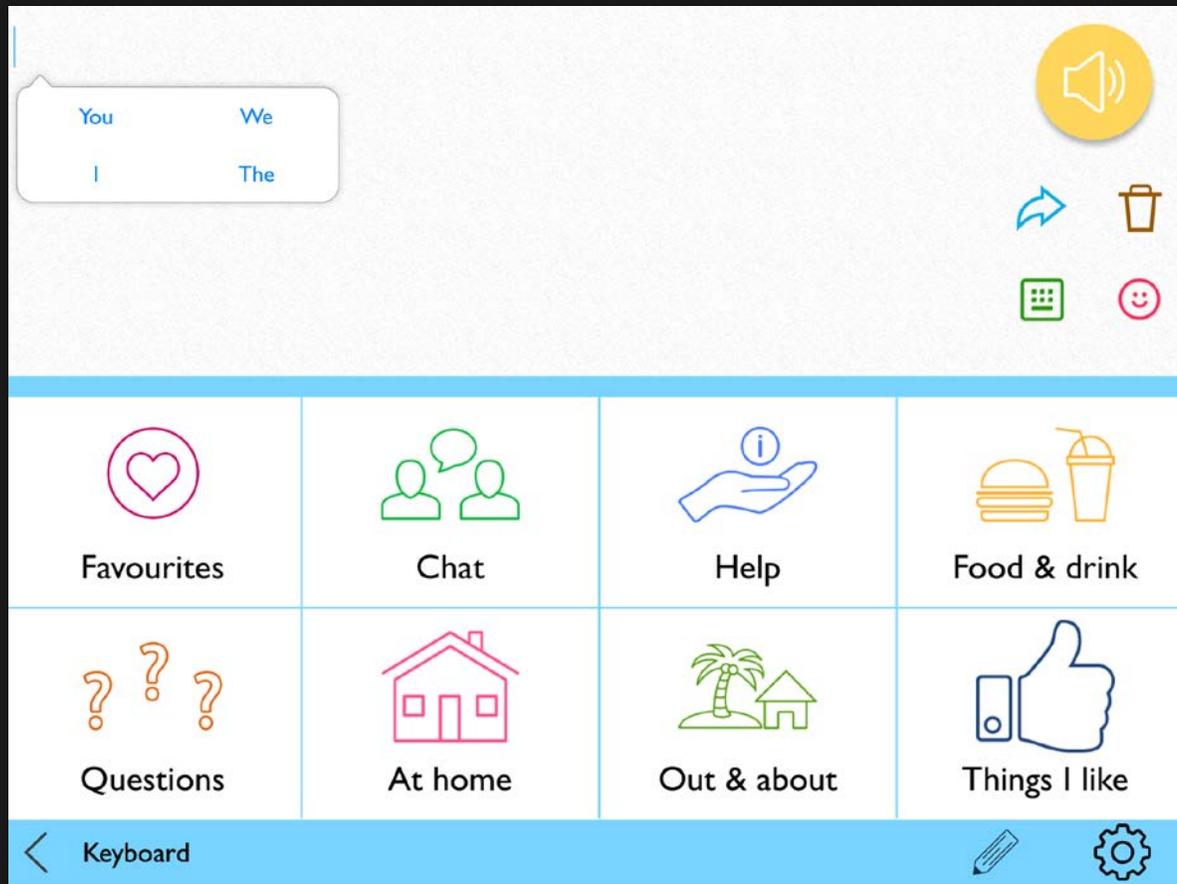


- Manage the call
- Greetings & Closings
- Repair breakdowns
- Share the main message

PHONE CHAT: ADDING A CATEGORY

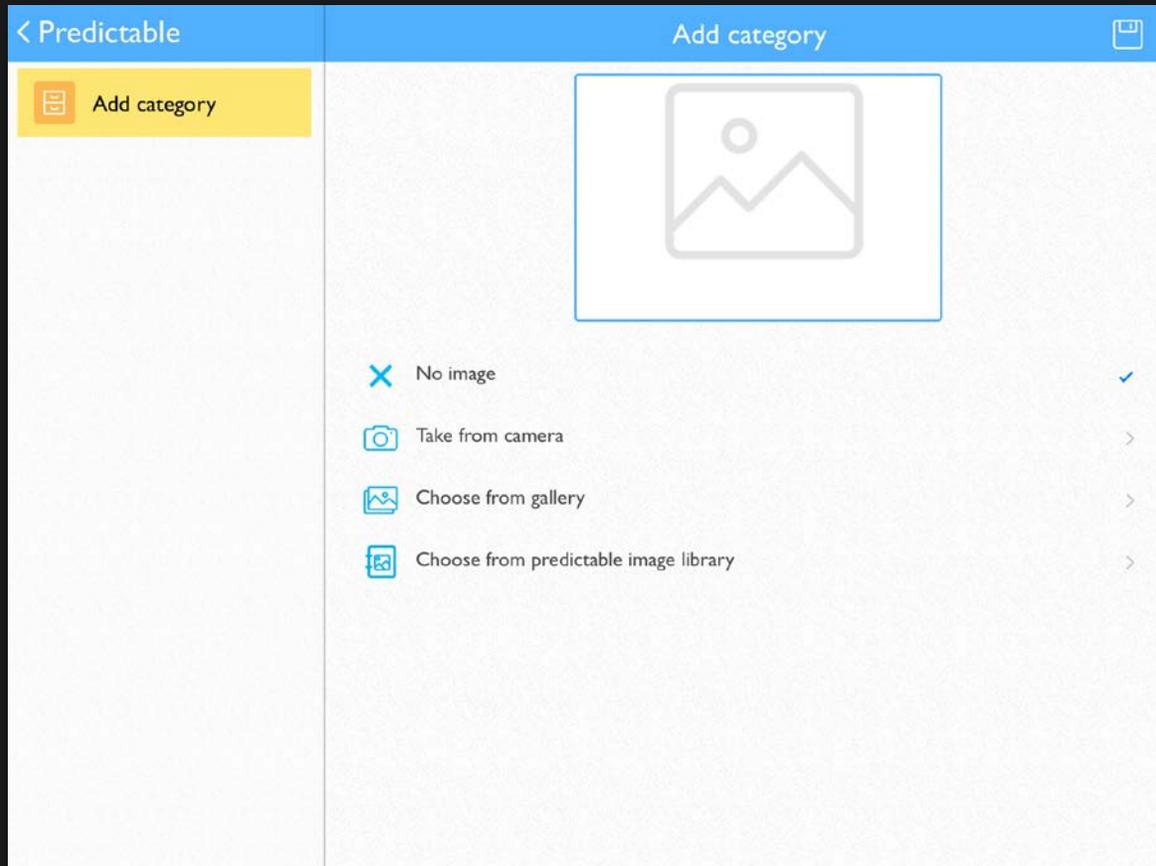


1. Tap the **Phrases Quick Key** icon.
2. Tap the **Pencil** icon.
3. Tap **Add Category**.
4. Type a **Category Name**.
5. Tap **Predictable**, then tap **Save**.

PHONE CHAT: ADDING A PHRASE

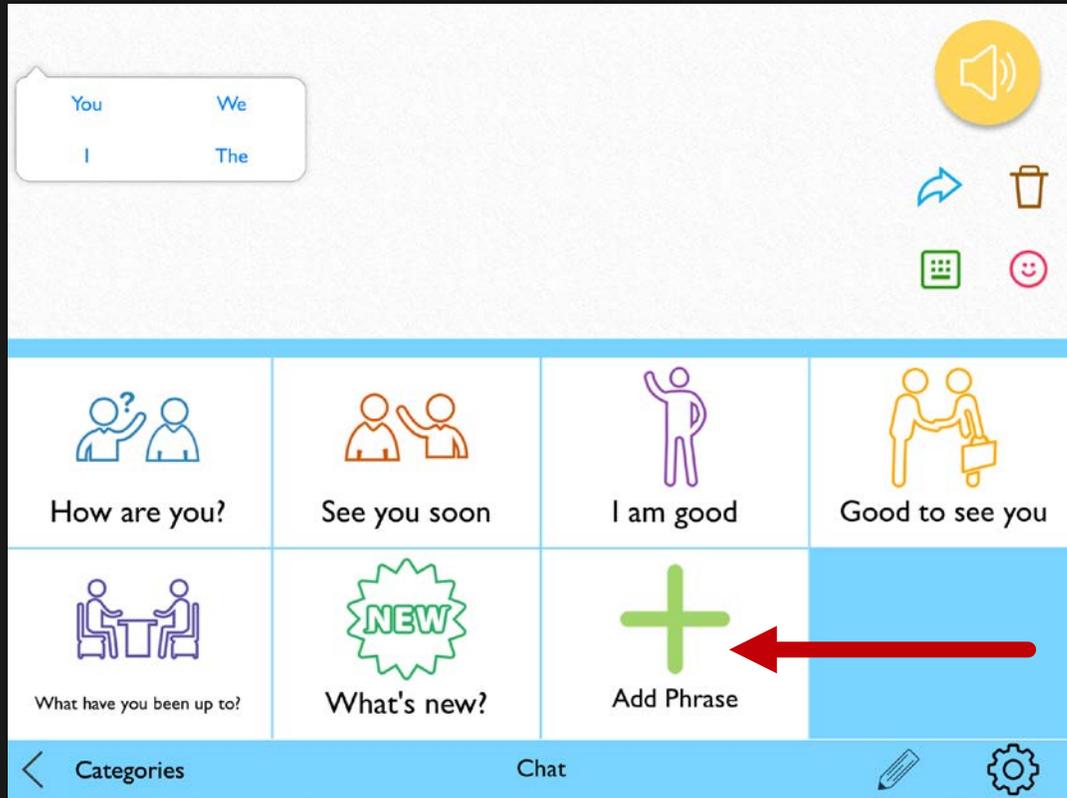


1. Tap **Phone Chat** category.
2. Scroll down or swipe to locate and tap **Add a New Phrase**.
3. Tap **Text**, then type in **Text to Speak** and **Text to Display**.
4. Add an **image** or **Audio** to the phrase.
5. Tap **Predictable** and then **Save** to save changes to the Phrase.

USE QUICK KEY: ADD PHRASE

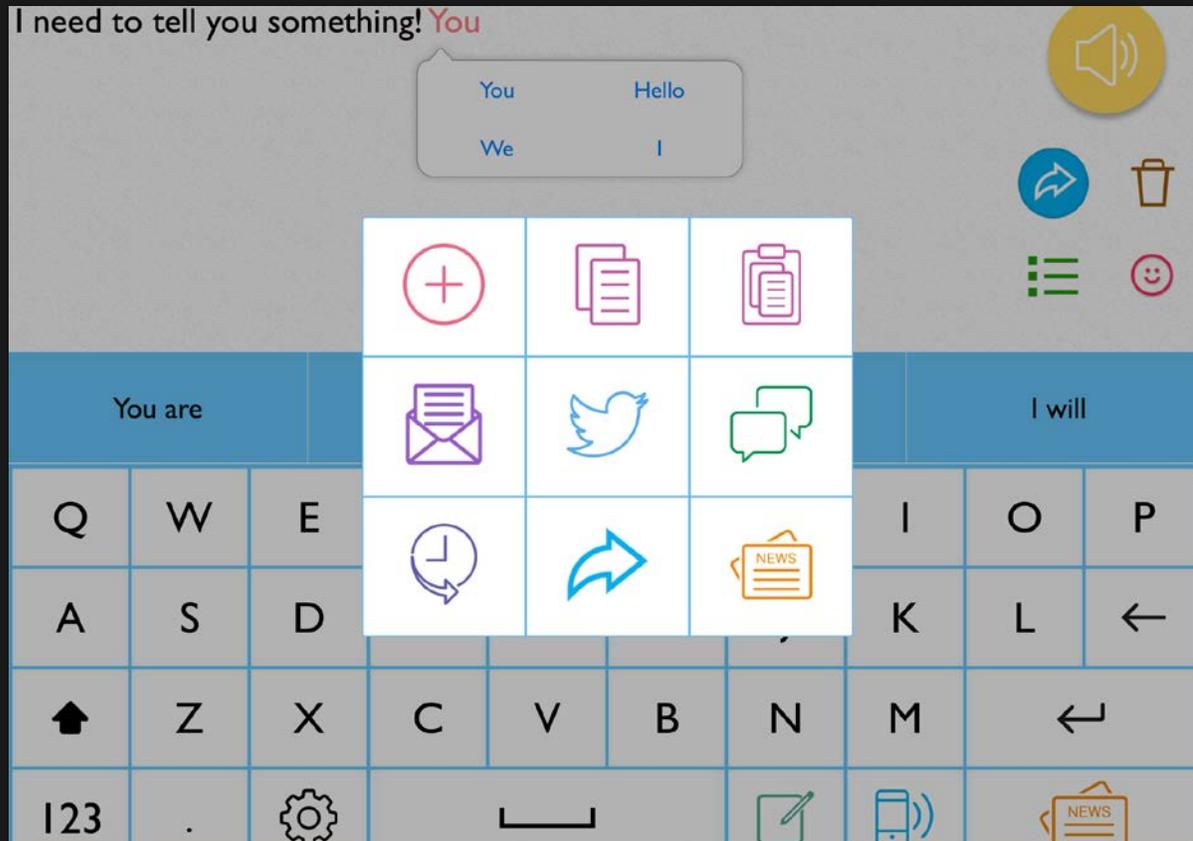


1. Type a message using the keyboard.
2. Tap Use icon.
3. Tap the pink Plus icon.
4. Select a category to store the phrase.
5. Customize by selecting the Text and Image menus.
6. Tap Predictable and then Save.

PHONE CHAT: MOVING A PHRASE OR CATEGORY

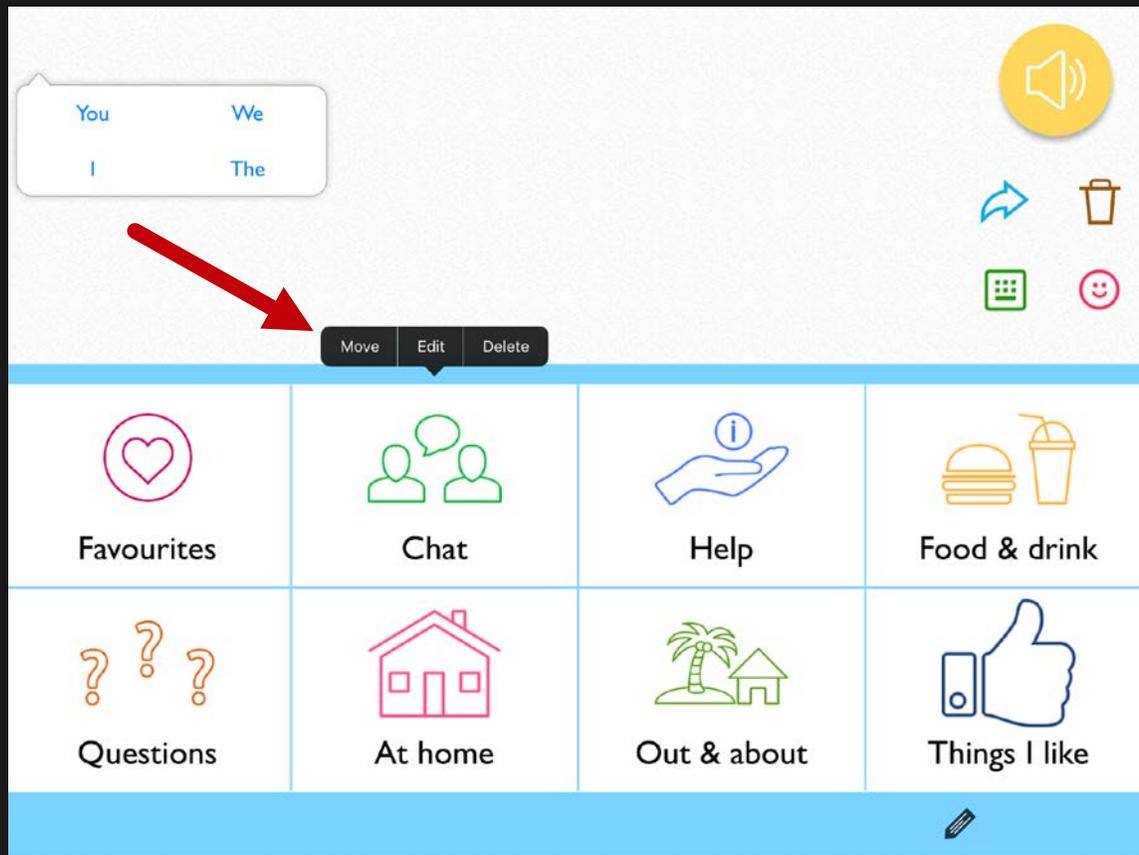

1. Tap the **Phrases Quick Key** icon
2. **Swipe** left or right to find and select your category
3. To move a phrase, tap the **Pencil** icon
4. Three options will appear: select **Move**
5. Press down and **drag** the icon where you want it
6. Select **Done**.


ADDING MEDIA TO A PHRASE

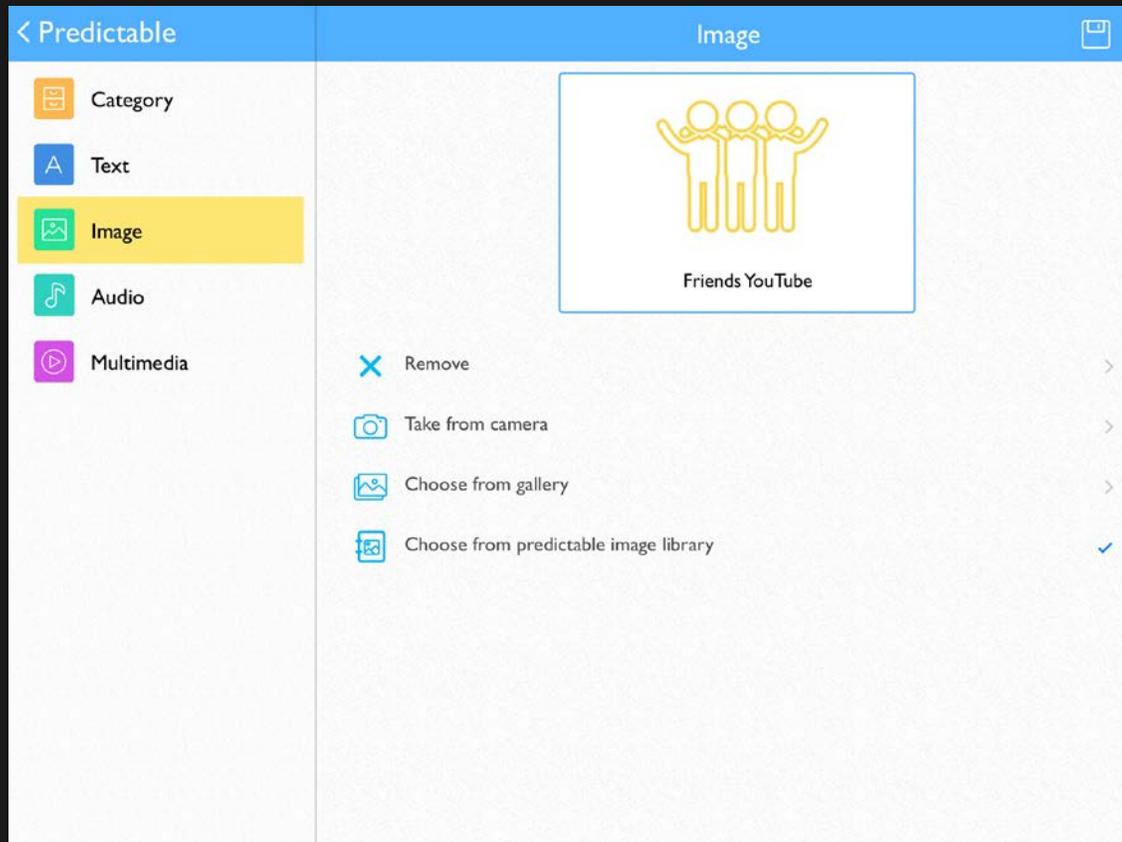

Add media after the phrase message speaks:

- Play recorded **Audio**
- Open a **YouTube** Video
- Open a **Website**


SENDING YOUR MESSAGE: EMAIL & SOCIAL MEDIA

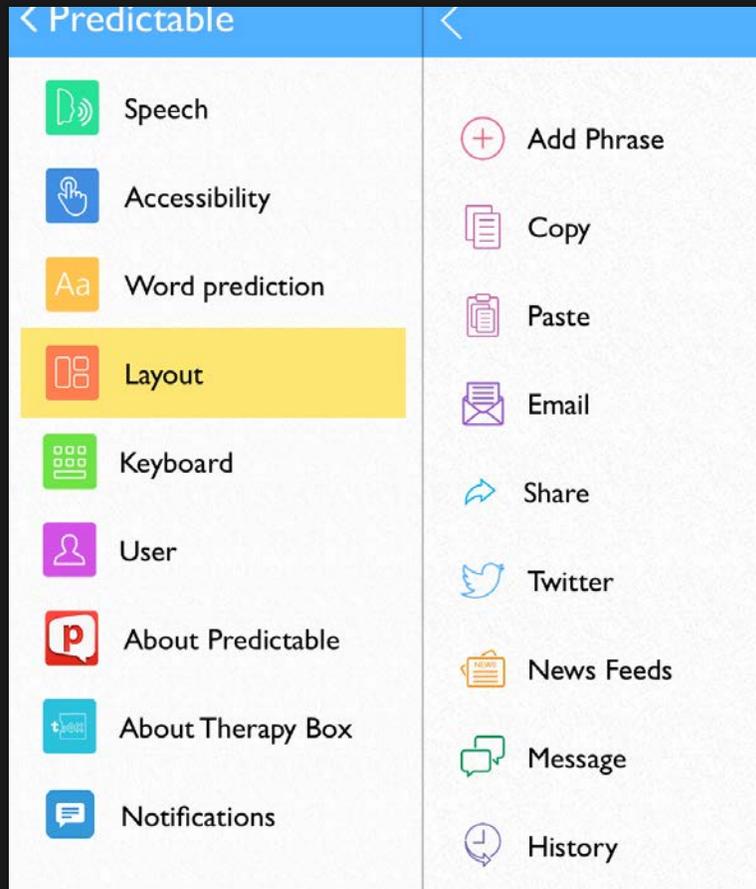

Send your message via email or a social media app.

- Selecting the **Use Quick Key**.
- Select **Share** (Twitter, Email, Facebook, Skype).
- Tap **News Feeds** to search current articles.

AUTOMATIC BACKUPS

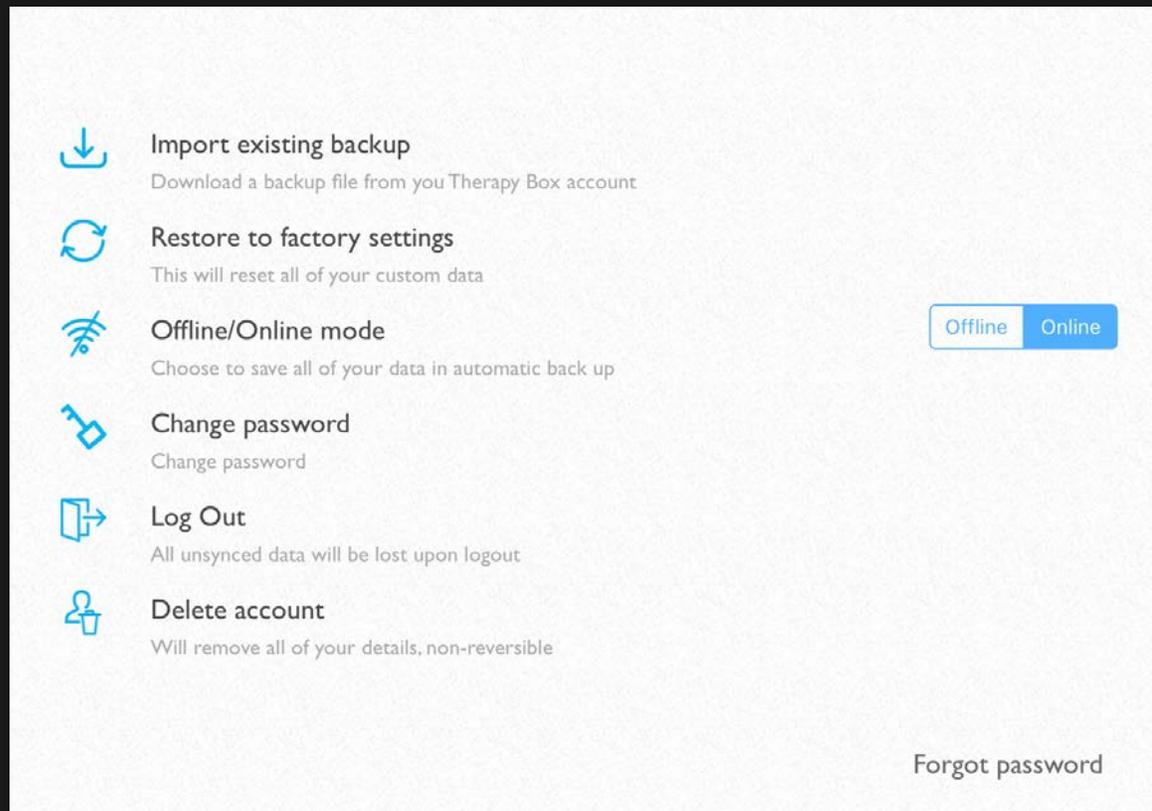

1. Tap on the **gear** icon.
2. Tap on **User**.
3. Select **Online** to enable automatic backups.

QUESTIONS??

Visit the Predictable website at <https://therapy-box.co.uk/apps/> for the most recent training videos and technical support articles

